

QualiSense GI CONDITIONER

Cavity Cleaning Agent

GI CONDITIONER is a mild polyacrylic acid solution designed to clean the cavity and to condition dentine and enamel, thus enhancing the bond between glass ionomer cement and tooth structure.

Contraindications

Do **not** use with patients with known allergic reactions to glassionomer cements. In case of an allergic contact contact to a physician.

Directions for Use

1. After having the tooth prepared apply **GI CONDITIONER** to the bonding surfaces for **20 seconds** by using a cotton pellet or brush.
2. Rinse thoroughly with water and dry gently. Do not desiccate!
3. Now the glassionomer cement can be applied.
4. Close the bottle immediately after use.

Warning

Do **not** allow the liquid to contact the oral tissues or skin. In case of contact, flush immediately with water.

Avoid contact with eyes. In case of contact, immediately flush with water while holding the eyelid open and seek medical advice.

Storage

Store **GI CONDITIONER** in a cool place at 4-25 °C (39-77 °F). Temperature should not exceed 25 °C (77 °F).

Warranty

DS Dental Supply GmbH warrants this product will be free from defects in material and manufacture. DS Dental Supply GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and DS Dental Supply GmbH's sole obligation shall be repair or replacement of the DS Dental Supply GmbH product.

Limitation of Liability

Except where prohibited by law, DS Dental Supply GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental use only!