

QualiSense DESENSITIZER

Instructions for use

DESENSITIZER is a silicone based one component acid protecting varnish with Nano-Fluorapatite, Nano-Calciumfluoride and soluble fluoride.

DESENSITIZER is for protection in cervical regions and free dentine areas after dental treatments. Nano-technology fluorapatite particles are known to enhance remineralization of tooth material.

DESENSITIZER does not require etching. It builds a protective film, that can be replenished, if necessary. The application is easy and time saving.

DESENSITIZER contains 2000 ppm (0,2 %) fluoride.

Indications

- application after bleaching procedures
- protection of hypersensitive cervicals
- protection after scaling, professional tooth cleaning

Application of DESENSITIZER

Clean the sensitive areas and rinse thoroughly. Gently dry in an oil- and water free air stream.

Before application vigorous shake the bottle with the **DESENSITIZER**. During shaking listen to the bead inside the bottle.

Apply **DESENSITIZER** 2 – 3 times in an even thin layer. Avoid contact of the **DESENSITIZER** with the mucosa.

Let **DESENSITIZER** dry for 1 minute or dry it gentle in an oil free air stream.

Important remarks

- In the first 2 hours after application of **DESENSITIZER** don't eat anything nor brush the teeth.
- Do not use **DESENSITIZER** under bridges and other restorative materials.
- The intense smell of **DESENSITIZER** volatizes during drying within 1 minute.
- **DESENSITIZER** contains ethylacetate. Ethylacetate is highly flammable. Keep away from sources of ignition. Avoid inhalation of ethylacetate vapour. After eye contact rinse out with plenty of water and call an ophthalmologist. After contact with skin, wash with plenty of water and soap.
- Impressions with vinyl silicones can be made earliest 4 weeks after the treatment with **DESENSITIZER**.
- For hygienic reasons the bottle containing **DESENSITIZER** is for single use and for one patient only.

Storage

Do not store above 25 °C (77 °F)!

Do not use after expiration date!

Store **DESENSITIZER** tightly closed in a well ventilated place.

Warranty

DS Dental Supply GmbH warrants this product will be free from defects in material and manufacture. DS Dental Supply GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and DS Dental Supply GmbH's sole obligation shall be repair or replacement of the DS Dental Supply GmbH product.

Limitation of Liability

Except where prohibited by law, DS Dental Supply GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep away from children!

For dental use only!