

## QualiSense SEAL VARNISH – SingleDose

### Instructions for use

**SEAL VARNISH** is a silicone based one component acid protecting varnish with Nano-Fluorapatite and Nano-Calciumfluoride for sealing of teeth. Nano-technology fluorapatite particles are known to enhance remineralization of tooth material.

**SEAL VARNISH** does not require etching. It builds a protective film, that can be renewed or repaired just as you like, if necessary. The application is easy and time saving.

**SEAL VARNISH** contains **2000 ppm** (0,2 %) fluoride.

**SEAL VARNISH** is designed for the following indications:

- Sealing of teeth where cleaning is difficult due to multi-band technique
- Sealing of teeth when patients are handicapped
- Temporary sealing of restorations with marginal leakage

### Application of SEAL VARNISH

The teeth are cleaned thoroughly with an oil free paste, polished and dried in an oil and water free air stream.

Before application vigorous shake the bottle with the **SEAL VARNISH**. During shaking listen to the glass bead inside the bottle.

Apply **SEAL VARNISH** 2 – 3 times in an even thin layer. Avoid contact of the **SEAL VARNISH** with the mucosa.

Let **SEAL VARNISH** dry for 1 minute or dry it gentle in an oil free air stream.

### Important remarks

- In the first **2 hours** after application of **SEAL VARNISH** don't eat anything nor brush the teeth.
- In case of protecting teeth with **SEAL VARNISH** during application of orthodontic multi-band technique, brackets are placed on the teeth before the application of **SEAL VARNISH**.
- The intense smell of **SEAL VARNISH** volatilizes during drying within 1 minute.
- **SEAL VARNISH** contains ethylacetate. Ethylacetate is highly flammable. Keep away from sources of ignition. Avoid inhalation of ethylacetate vapour. After eye contact rinse out with plenty of water and call an ophthalmologist. After contact with skin, wash with plenty of water and soap.
- Impressions with vinyl silicones can be made earliest 4 weeks after the treatment with **SEAL VARNISH**.
- For hygienic reasons the bottle containing **SEAL VARNISH** is for single use and for one patient only.

### Storage

Do **not** store above **25 °C (77 °F)**!

**Do not use after expiration date!**

Store **SEAL VARNISH** tightly closed in a well-ventilated place.

### Warranty

DS Dental Supply GmbH warrants this product will be free from defects in material and manufacture. DS Dental Supply GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and DS Dental Supply GmbH's sole obligation shall be repair or replacement of the DS Dental Supply GmbH product.

### Limitation of Liability

Except where prohibited by law, DS Dental Supply GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**Keep away from children!**

**For dental use only!**